Gutta Percha Points

 **META BIOMED**

2024. 10. 24. DM-119-5

SYMBOLS

 Manufacturer	 Date of manufacture	 Temperature limit	 Do not re-use
nl Fabrikant fr Fabricant de Hersteller it Produttore pt Fabricante es Fabricante tr Üretici el Κατασκευαστής	nl Productiedatum fr Date de fabrication de Herstellungsdatum it Data di produzione pt Data de fabrico es Fecha de fabricación tr Üretim tarihi el Ημερομηνία κατασκευής	nl Temperatuurlimiet fr Limites de la température de Temperaturgrenzwerte it Limite di temperatura pt Limitação de temperatura es Limitación de temperatura tr Sıcaklık sınırı el Περιορισμός θερμοκρασίας	nl Niet hergebruiken fr Ne pas réutiliser de Nicht zur Wiederverwendung it Non riutilizzare pt Não reutilizar es No reutilizar tr Yeniden kullanmayın el Αποφύγετε η επαναχρησιμοποίηση
 Batch Code	 Use-by date	 Caution	 Keep away from sunlight
nl Partijcode fr Code de lot de Chargennummer it Codice Lotto pt Código de lote es Código de lote tr Parti Kodu el Κωδικός παρτίδας	nl Gebruiken vóór fr Utiliser avant de Verwenden bis it Uso previsto pt Usar até es Usado por tr Kullanım el Χρήση από	nl OPGELET fr PRÉCAUTIONS de Achtung it Attenzione pt Cuidado es Cuidado tr İkaz el Προσοχή	nl Uit direct zonlicht houden fr Tenir à l'écart de la lumière directe du soleil de Vor Sonneneinstrahlung schützen it Tenere lontano dalla luce solare diretta pt Manter longe da luz solar direta es Mantener alejado de la luz solar directa tr Doğrudan güneş ışığından uzak tutun el Κρατήστε το μακριά από το ηλιακό φως

 **CE - Mark and identification number of notified body. Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC**

 **Authorized representative in the European Community / European Union**

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 **Manufacturer**

META BIOMED CO., LTD.

270, Osongsaengmyeong 1-ro,
Osong-eup, Heungdeok-gu, Cheongju-si,
Chungcheongbuk-do, Republic of Korea

Tel +82-43-216-0433
Fax +82-43-217-1988
E-Mail info@meta-biomed.com
www.meta-biomed.com

 **Meta Biomed Europe GmbH**

Schederhofstraße 59, 45145 Essen, Germany

Tel +49-208-30991910
Fax +49-208-30991999

 **INSTRUCTION FOR USE**

Gutta percha points are root canal obturation material. It is used to fill the cleaned, shaped and irrigated root canal space.

- Time saving with easy filling
- For faster and easier root canal obturation
- Variable tapers & sizes
- Length marking available
- Excellent radiopacity
- Long shelf life and freshness assured
- Suitable for vertical and lateral condensation

INDICATION AND PURPOSE

- ① Gutta percha point is an integral material in promoting periapical healing and preventing disease progression.
- ② Endodontic therapy system using the gutta percha point obturated impermeable to liquid and bacteria

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STORAGE
Store at room temperature of 1°C-30°C (33°F-86°F)

SHELF LIFE
4 years form manufacture date

AVAILABLE SIZE

Type	Sizes
Standardized (ISO Specifications)	10, 15, 20, 25, 30, 35, 40, 45, 50, 55, 60, 70, 80, 90, 100, 110, 120, 130, 140
Non-Standardized (Accessory Sizes)	X-Fine, Fine-Fine, Medium-Fine, Fine, Fine-Medium, Medium, Medium-Large, Large, X-Large, F1/F2/F3
Special Taper (.04, .06, .08)	15, 20, 25, 30, 35, 40, 45, 50, 55, 60, 70, 80
Japanese type	JF-S, JF-M, JF-L, JS, JM, JL, JL-S, JL-M, JL-L - De Trey Type : XX-F, X-F, Fine, Medium

COMPONENTS

- Available packaging

- ① 6 compartment dispenser - Code: P.21/ Regular packing points: 120 points
- ② Sliding type dispenser - Code: P.71/ Regular packing points: 120 points
- ③ 6 Vial packing (Square) - Code: P.31/ Regular packing points: 120 points
- ④ 6 Vial packing (Round) - Code: P.31/ Regular packing points: 120 points
- ⑤ 4 compartment dispenser - Code: P.51/ Regular packing points: 100 points
- ⑥ Endo Organizer (Spill Proof 12 Compartments) - Code: P.11/ Regular packing points: 240points/ Size: ISO 15-80

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