

# Meta Etchant

Meta Etchant contains 37 weight percent phosphoric acid in water thickened with unique polymer

## INSTRUCTION FOR USE

### Meta Etchant contains 37 weight percent phosphoric acid in water thickened with unique polymer

**Meta Etchant** is gel for enamel and dentin conditioning for the sealing and direct composite restorations, adhesive cementation of inlays, crowns, bridges veneers, adhesive bridges and bonding of orthodontic attachments.

#### INSTRUCTIONS

- ① **Isolation:** Rubber dam is the preferred method for isolation.
- ② **Cavity preparation:** Prepare cavity with minimal tooth reduction.
- ③ **Tooth conditioning:**  
**Dentin pretreatment:**  
Apply 37% **Meta Etchant** gel to available enamel/dentin for 15 seconds.
- ④ **Rinse and Blot dry:** Remove the gel by suction using aspirator tube and rinse for 20 seconds. Blot dry conditioned area with a moist cotton pellet.

#### SIDE EFFECTS

- None known to date

#### WARNING

- Meta Etchant is corrosive and contains Phosphoric acid. Protective eyewear is recommended when using **Meta Etchant**. Avoid contact with eye and skin. If accidental eye contact occurs, flush immediately with water and consult a physician.



CORROSIVE

#### CAUTIONS

- For dental use only.
- Keep away from child.
- Do not use Meta Etchant after date of expiration.

#### STORAGE

- Store at 18°C~24°C(64°F~75°F)
- Refrigeration is recommended to extend shelf life

#### SHELF LIFE

- 2 years from the manufactured date.

#### COMPONENTS

- Meta Etchant 3g x 3 syringes
- Disposable stainless tips 20 ea

#### SYMBOLS

Use By

Batch Code

CE-Mark and identification number of notified body. Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC

Attention, See instruction for use

Manufacturer

Made in Korea

Manufactured by  
**META BIOMED CO.LTD.**

634, Yeonje-ri, Gang-oe-myeon, Cheong-won-gun  
Chungcheongbuk-do, Korea  
Tel: +82-43-216-0433  
Fax: +82-43-217-1988  
E-mail: info@meta-biomed.com  
Web site: http://www.meta-biomed.com

**META BIOMED**

DM-21-5

## CHECKING SHEET

Meta Etchant

TITLE	
CODE	DM-21-5
SIZE	155x181mm
PAPER	60g 모조
MAIN COLOR	INDIGO 2767C 별색 단도
SPECIAL	
FUNCTION	
PRINT COLOR	



INDIGO  
2767C

DATE	2009. 07. 10.	
DESIGNER	진희남	
RECOGNITION	김상일	
CONFIRMATION	<b>META BIOMED CO.LTD.</b>	
	DATE	NAME
		SIGN

2011. 06. 02 → DM-21-5 / 주소변경 (홍선미)