

# Metapaste

Calcium Hydroxide  
with Barium Sulfate

## Water Soluble Temporary Root Canal Filling Material

### APPLICATIONS

- Exposed pulp in pulp capping and pulpotomy
- Leakage canal
- Apexification
- Formation of hard tissue barrier
- Root canal filling materia

### DIRECTIONS FOR USE

- Clean and dry it, after preparing the root canal
- Set up a disposable tip in syringe and insert the ring provided for an easy re-direction of the tip
- Disinfect the disposable tip with ethanol. Fill the canal completely by pushing the syringe plunger while withdrawing the tip slowly
- Clean any excess paste with a sterilized cotton pellet
- Remove the used tip, fix a new one and cover up

### How to use Rotary Ring

- If contact with air, as content could be hardened, so it is necessary to cover a cap on the needle after use
- Putting a Syringe into a Rotary Ring os more convenient for use to handle & Use Rotary Ring as a wrench to tighten or loosen a tip

### ADVANTAGES

- Easy cleaning and removal
- Excellent antibacterial effect and radiopacity
- Premixed paste in a convenient syringe
- Excellent accessibility to the root canal and prevention of cross-contamination

## INSTRUCTION FOR USE

### CONTENTS

- 2,2g paste in a syringe
- Disposable tips
- One ring rotator for the direction control of the tip



### STORAGE

- Do not fill beyond the apex
- For dental use only
- Do not use in allergic person
- Keep the syringe capped at all times, when not in use
- Store in a dry and cool area to avoid dryness or separation of the paste

### SHELF LIFE

- 2 years from the manufacturing date

### SYMBOLS

- Use By
- Lot Code
- CE-Mark and identification number of notified body. Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC
- Attention, See instruction for use
- Manufacturer

Made in Korea

Manufactured by  
**META BIOMED CO.LTD.**

634, Yeonje-ri, Gang-gae-myeon  
Cheongwon-gun, Chungcheongbuk-do, Korea  
Tel: +82-43-216-0433  
Fax: +82-43-217-1988  
E-mail: info@meta-biomed.com  
Web site: http://www.meta-biomed.com



DM-10-5

## CHECKING SHEET

Metapaste

TITLE	
CODE	DM-10-5
SIZE	155x175mm
PAPER	60g 무조
MAIN COLOR	INDIGO 2767C 별색 단도
SPECIAL	
FUNCTION	
PRINT COLOR	



DATE	2009. 07. 10.		
DESIGNER	진희남		
RECOGNITION	김상일		
CONFIRMATION	META BIOMED CO.LTD.		
	DATE	NAME	SIGN

2011.05.23 DM-10-5 주소변경 (홍선미)