Absorbent Paper Points

Sterile Absorbent Paper Points

 **META BIOMED** 2025. 02. 03. DM-124-6

SYMBOLS

Manufacturer

nl Fabrikant
fr Fabricant
de Hersteller
it Produttore
pt Fabricante
es Fabricante
tr Üretici
el Παρασκευαστής

Batch Code

nl Partijcode
fr Code de lot
de Chargennummer
it Codice Lotto
pt Código de lote
es Código de lote
tr Parti Kodu
el Κωδικός Παρτίδας

Temperature limit

nl Temperatuurlimiet
fr Limite de température
de Temperaturbegrenzung
it Limite temperatura
pt Limite de temperatura
es Limite de temperatura
tr Sıcaklık limiti
el Όριο Θερμοκρασίας

Keep away from sunlight

nl Uit direct zonlicht houden
fr Tenir à l'écart de la lumière directe du soleil
de Vor Sonneneinstrahlung schützen
it Tenere lontano dalla luce solare diretta
pt Manter longe da luz solar direta
es Mantener alejado de la luz solar directa
tr Doğrudan güneş ışığından uzak tutun
el Κρατήστε τα μακριά από το άμεσο ηλιακό φως

Date of manufacture

nl Productiedatum
fr Date de fabrication
de Herstellungsdatum
it Data di produzione
pt Data de fabrico
es Fecha de fabricación
tr Üretim tarihi
el Ημερομηνία παρασκευής

Use-by date

nl Houdbaarheidsdatum
fr Date limite d'utilisation
de Verwendbar bis
it Data di scadenza
pt Data de validade
es Fecha de caducidad
tr Son kullanim tarihi
el Ημερομηνία λήξης

Consult instructions for use

nl Raadpleeg de gebruiksaanwijzing
fr Consulter le CONSIGNES D'UTILISATION
de Gebrauchsanweisung beachten
it Consultare le istruzioni per l'uso
pt Consulte as instruções de uso
es Consulte las instrucciones de uso
tr Kullanım Talimatına Başvurun
el Συμβουλευτείτε τις οδηγίες χρήσης

Sterilized using irradiation

nl Gesteriliseerd door middel van bestraling
fr Stérilisé par irradiation
de Sterilisation durch Bestrahlung
it Sterilizzato mediante irradiazione
pt Esterilizado com uso de radiação
es Esterilizado mediante irradiación
tr Irradyasyon kullanılarak sterilize edilmiştir
el Αποστείρωση με γ-ακτινοβολία

CE - Mark and identification number of notified body.
1639 Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC

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EC REP Authorized representative in the European Community / European Union

nl Gemachtigde vertegenwoordiger in de Europese Gemeenschap / Europese Unie
fr Représentant autorisé dans la Communauté européenne / Union européenne
de Bevollmächtigter in der Europäischen Gemeinschaft / Europäischen Union
it Rappresentante autorizzato nella Comunità Europea / Unione Europea
pt Representante autorizado na Comunidade Europeia / União Europeia
es Representante autorizado en la Comunidad Europea / Unión Europea
tr Avrupa Topluluğundaki / Avrupa Birliğindeki Yetkili temsilci
el Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα/ Ευρωπαϊκή Ένωση

Do not resterilize

nl Niet opnieuw steriliseren
fr Ne pas restériliser
de Nicht erneut sterilisieren
it Non sterilizzare di nuovo
pt Não voltar a esterilizar
es No lo vuelva a esterilizar
tr Tekrar sterilize etmeyin
el Μην αποστείρωνετε ξανά

Manufacturer
META BIOMED CO., LTD.
270, Osongsangmyeong 1-ro,
Osong-eup, Heungdeok-gu, Cheongju-si,
Chungcheongbuk-do, Republic of Korea
Tel +82-43-216-0433
Fax +82-43-217-1988
E-Mail info@meta-biomed.com
www.meta-biomed.com

EC REP

Meta Biomed Europe GmbH

Schederhofstraße 59, 45145 Essen, Germany
Tel +49-208-30991910
Fax +49-208-30991999

en INSTRUCTION FOR USE

Sterile Absorbent Paper Points

*Handrolled for extra absorbency

- Highly absorbent
- Length marked available
- Variety sizes and packages available
- Color coded

INDICATION AND PURPOSE

Paper points are used to dry the root canal before obturation and to clean the root canals that are to be sealed in endodontic treatments.

INSTRUCTION FOR USE

1. Measure the length of root canal
2. The formation of the root canal of the same size as the last file to select
3. The root canal using paper points to dry
4. About 10 ~ 15 points is used per a root canal
5. Sealer or cement is applied by using paper points

CAUTIONS

1. Should be use sterilized paper points
2. Do not use bent or broken points
3. Should be careful to avoid overfilling or dead space when applied sealer or cement
4. Use it immediately when applied sealer or cement
5. After using the paper points, should be kept in a cool place
6. Do not expose to direct sunlight
7. Do not reuse to prevent cross infections. (Secondary infections such as pathogens and viruses caused by saliva and blood)

STORAGE

Store at room temperature of 1°C~30°C(33°F~86°F)

SHELF LIFE

5 years form manufacture date

AVAILABLE SIZE

Type	Sizes
Standardized (ISO Specifications)	10, 15, 20, 25, 30, 35, 40, 45, 50, 55, 60, 70, 80, 90, 100, 110, 120, 130, 140
Non-Standardized (Accessory Sizes)	XX-Fine, X-Fine, Fine, Medium, Coarse, X-Coarse, Protaper Size(F1~F5)
Special Taper (04, 06, 08)	15, 20, 25, 30, 35, 40, 45, 50, 55, 60, 70, 80

COMPONENTS

- Available packaging

1. 6 compartment dispenser - Code: P.21 / Regular packing points: 60 / 200 points
2. Sliding type dispenser - Code: P.71 / Regular packing points: 60 / 200 points
3. 6 Vial packing(Square) - Code: P.31 / Regular packing points: 60/200 points
4. 6 Vial packing(Round) - Code: P.31 / Regular packing points: 60/200 points
5. 4 compartment dispenser - Code: P.51 / Regular packing points: 200 points
6. Blister Cell - Code: P.61 / Regular packing points: 180/200 points

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